

AMIeyes

www.fillyourskin.com



Regenerate your skin with AMI EYES

AMI EYES is a premium PN product which improves the conditions of damaged dermis with tissue regeneration material and purified DNA from salmon milt.



www.fillyourskin.com

Safe Ingredient: PN

PN is the safe material for human compatibility approved by the Ministry of Food and Drug Safety.



High Elasticity

AMI eyes recovers and improves the damaged and aged skin. It strengthens the skin barrier by filling vital components.



Recovery

AMI eyes activates self regeneration and improves the skin problem like scars, wrinkles and so on.



SALMON



PN



INJECTION

www.fillyourskin.com

PN | Poly-nucleotide

Polydeoxyribonucleotides (PDRN) has previously been used as a skin injectable for anti-aging and hair growth treatments in Korea, and medically for skin regeneration purposes.

PN is a purer and more concentrated form of PDRN and has stronger regenerative properties.

Skin elasticity
Increases skin elasticity



Shrink pores
Tightening effect of enlarged pores



improvement of skin problems
Increase fibroblast function



Skin moisturizing
Abundant moisture supply with polymer hyaluronic acid



Skin tone & skin texture improvement
Improvement of skin tone and oil-water balance



www.fillyourskin.com

AMI EYES specification

- 1% (20mg) Poly-nucleotide
- One pre-filled syringe of 2mL
- Country of origin in South Korea
- Storage in Room temperature

● Under eyes
 --- Meso therapy



AMI EYES FOR

www.fillyourskin.com

Under eye treatment



1. Use 30g 13mm needle or Multi needle / Meso needle
 2. Point 6~10 spots under one eye
 3. Inject the same amount (1mL for one eye / 2mL for one session total)
- * You can inject more amount on the dark circle areas.

Mesotherapy



1. Use 33g needle or Multi needle / Meso needle
2. Inject the same amount (2mL for whole face)


* If you use AMI EYES with Colladerm or HA booster, the effect will be better and last longer.



www.fillyourskin.com

Before & After

Intensive treatment gives your skin brightness and glow.

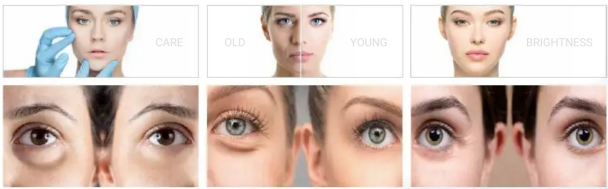

1ST SESSION

3 - 5 DAYS AFTER

- Skin texture improvement
- Sebum secretion reduced

2ND SESSION

2 - 4 WEEKS AFTER

- Increased skin elasticity
- Self-generating collagen
- Pore size reduced

3RD SESSION

4 - 6 WEEKS AFTER

- Natural lifting effect
- Skin volumizing effect

www.fillyourskin.com



Moisturizing

Skin tone

Pore Improvement

Regeneration



www.fillyourskin.com