

## **Bio-Stimulating Collagen Booster**

### **What is this product?**

- **Biodegradable collagen booster**
- **Active ingredient:** To be updated after finalizing the formula
- **Unit:** 200mg
- **Particle size:** 30-60  $\mu\text{m}$
- **Injection layer:** Dermis
- **Expected duration of effect:** About 18 months
- **Product hydration:** Sterile distilled water or saline solution (5ML / 8ML)
- **Needle:** 23-25G (May vary between 18G-34G based on the medical professional's choice)
- **Cannula:** 4.0~5.0mm
- **Repeat treatments:** For optimal results, administer three sessions with a one-month interval

### **Effects of the product:**

- Skin volume enhancement
- Natural contouring
- Correction of deep and fine wrinkles
- Acne scars improvement
- Stretch mark reduction
- Improvement of skin elasticity

## Usage Instructions

Reconstitution Guide 1		
(Manual Mixing)	Add 8ml of sterile distilled water or saline solution 24 hours before use or immediately before use.	
Reconstitution Guide 2		
(Using a Mixer)	<b>Mixer Type:</b> Vortex Mixer	<b>Speed:</b> 3,000 RPM
RECONSTITUTION WITH		
Reconstitution with	Sterile distilled water or saline solution	Volume: 5ml, 8ml

### Product Preparation

- Use the mixed solution within **24 hours** when refrigerated; do not reuse after storage.
- Single-use product.**

## For Medical Professionals

- **Hydration:** 5ml / 8ml
- **Needle:** 23-25G
- **Cannula:** 4.0~5.0mm
- **Injection layer:** Dermis (**For acne or scars, inject shallowly.**)
- **Anesthesia Options:**
  - General anesthesia
  - Local anesthesia
  - **Lidocaine (1ml or 2ml)** can be mixed with the product (adjust the amount of sterile distilled water or saline accordingly)

## Preventing Undesired Side Effects

### Customer Guidelines:

- Take **prescribed antihistamines and anti-inflammatory medications** regularly.
- Inform your doctor of your medical history to **avoid adverse effects.**
- Minor **bruising and swelling** may occur but should subside over time.
- **Avoid excessive exercise and alcohol consumption on the day of treatment.**
  - **(Sauna and shower are allowed / Makeup is allowed the next day)**
- **Avoid direct sunlight and UV exposure** until redness and swelling subside. *(Use sunscreen.)*
- **Massage the treated area for about a week** after the procedure. *(Avoid excessive rubbing or impact.)*