



Miracle

MIRACLE

SERIES

Always stay new



What is Miracle?

A new type of Meso-ampoule for the Collagenesis.
Strong Collagen Rejuvenator for fundamental anti-aging care.




Miracle rejuvenates our skin through the Collagenesis on the entire face.
It stimulates skin lifting and restores skin elasticity.



CESABP Technology

Collagenesis-Enabled Solubilized Active and Biodegradable Polymer

competitor

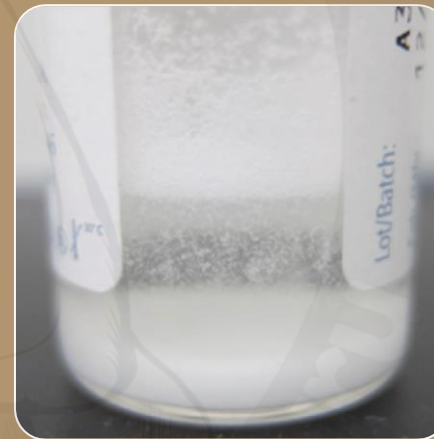

Polymer microparticle+carrier

Miracle



Fully Solubilized PCL

Miracle L



Solubilized PCL

- Collagenesis
- Entire face lifting
- Elasticity & Density

Miracle H



Solubilized PCL
+
HA

- Collagenesis
- Entire face lifting
- Elasticity & Density
- Instant hydration



World's Patents

World's unique technology of manufacturing of Solubilized PCL for natural skin collagen synthesis



Patent No.(KR)
10-2018-0058947





Patent No.(US)
16/410,611



Patent No.(PCT)
KR2018/005988



Mechanism

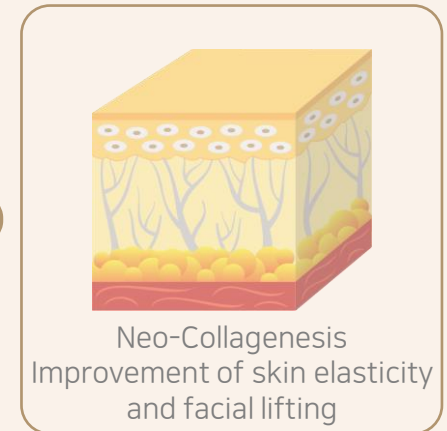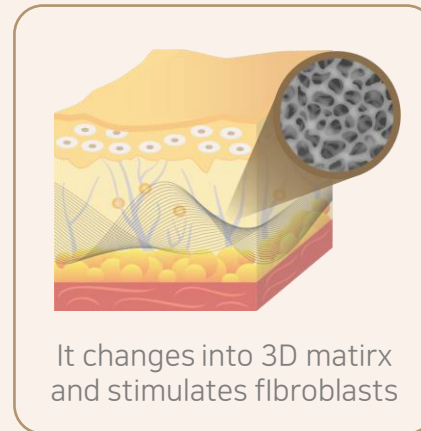





Clinical Data

In Vivo test



PBS (Positive Control)

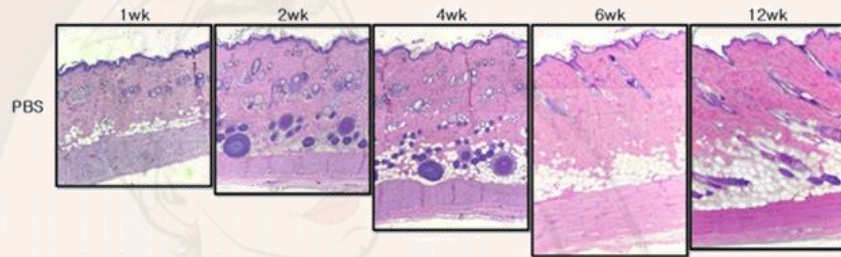

MIRACLE

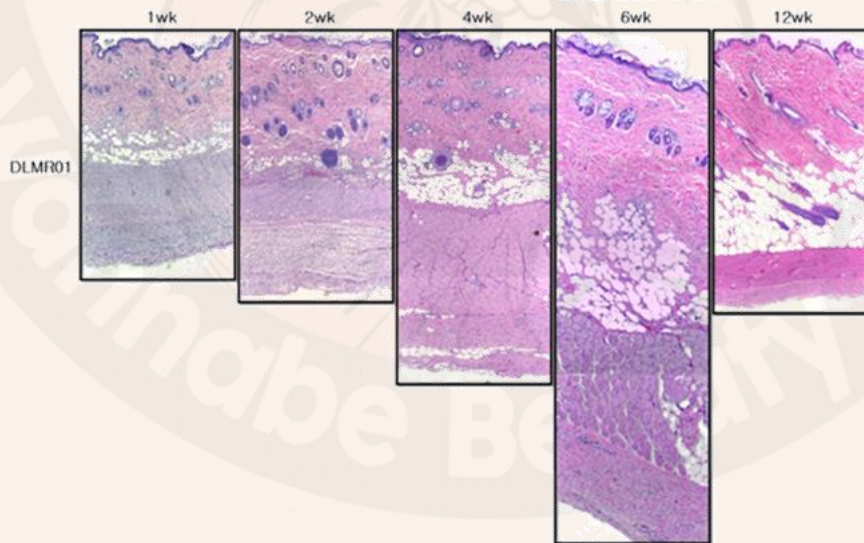

Before & After

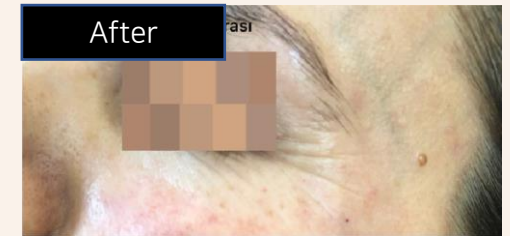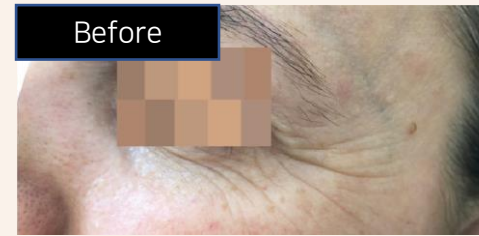



Protocol



1. Cleanse well in lukewarm water.
2. Apply Lidocaine cream to target area.
3. Leave about 10 minutes.
4. Cleanse cream with alcohol cotton.
5. Apply 0.02cc ~ 0.05cc each point evenly to skin.

**Tip*

- ✓ *Application depth: 0.5~1mm(superficial dermis) / 30G needle*
 - ✓ *Escape eyebrow part.(It can shed blood a lot)*
 - ✓ *Do not mix with other product.*
6. Carefully cleanse skin with alcohol cotton.
 7. Evenly apply Post later(treatment) cream to application part.



Precautions for Use

- ✓ Avoid exposure to heat and radiation (sauna, sun) swimming/bathing and alcohol consumption for the first 24 hours.
- ✓ An anesthetic cream or local anesthetic injection can be given before the procedure at the discretion of the doctor.
- ✓ Avoid Blood Vessels while injecting and take special care when you treat under eyes.
- ✓ If you have a patient with allergy or Irritable Syndrome, we recommend taking Antihistamines(Prescription-Free) before Treatment.
- ✓ Not recommended mixing with other products.



Storage and After Use

- ✓ Store at room temperature up to 30°C and recommend using it all on the same day if possible after opening.
- ✓ After use, make sure to dispose of it.



A
Drop
of
Miracle



ALWAYS STAY NEW

The World's Unique Technology
Fully Solubilized PCL

www.dexlevo.com



Miracle