

HYALSKIN

REDJUR[®] eyes

Perfect solution for Crow's feet



RED BIO COMPANY

What is REDJUR eyes?



REDJUR eyes has low viscosity, which makes it more suitable for thin and sensitive skin around the eyes. It also contains **Polynucleotide(PN)** - a DNA molecule extracted from salmon gonads that improves skin wellness and boosts the restoration of youthfulness for a more natural result.

[*Polynucleotide Eye Booster _ REDJUR eyes](#)

Main Ingredient	High-Quality PN (Polynucleotide) 1%
Volume	2.0mL x 1syr
Benefit	<ul style="list-style-type: none"> Reduces dark circles and fine lines Boosts elasticity and hydration Brightens and refreshes eyes Non-surgical eye rejuvenation




REDJUR® eyes

→ PN (Polynucleotide Eye Booster)


→ Redjur Eyes PN 1% is a premium eye rejuvenation solution formulated with high-purity Polynucleotide (PN) at 1% concentration, specifically designed for the delicate under-eye area.



"Skin healing care is essential to restore and strengthen thin, damaged inner skin."

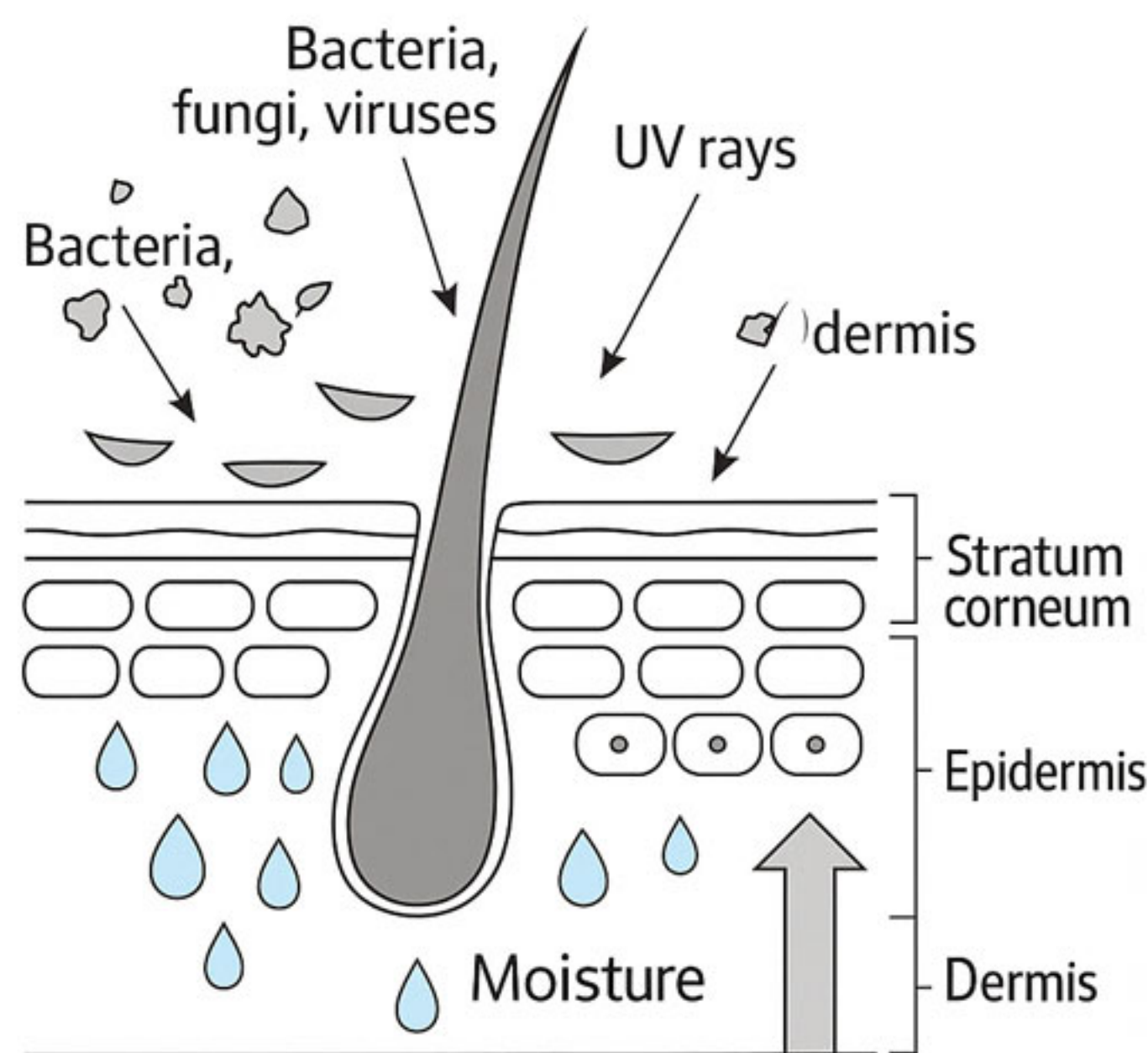


Thinned dermis with again and damage

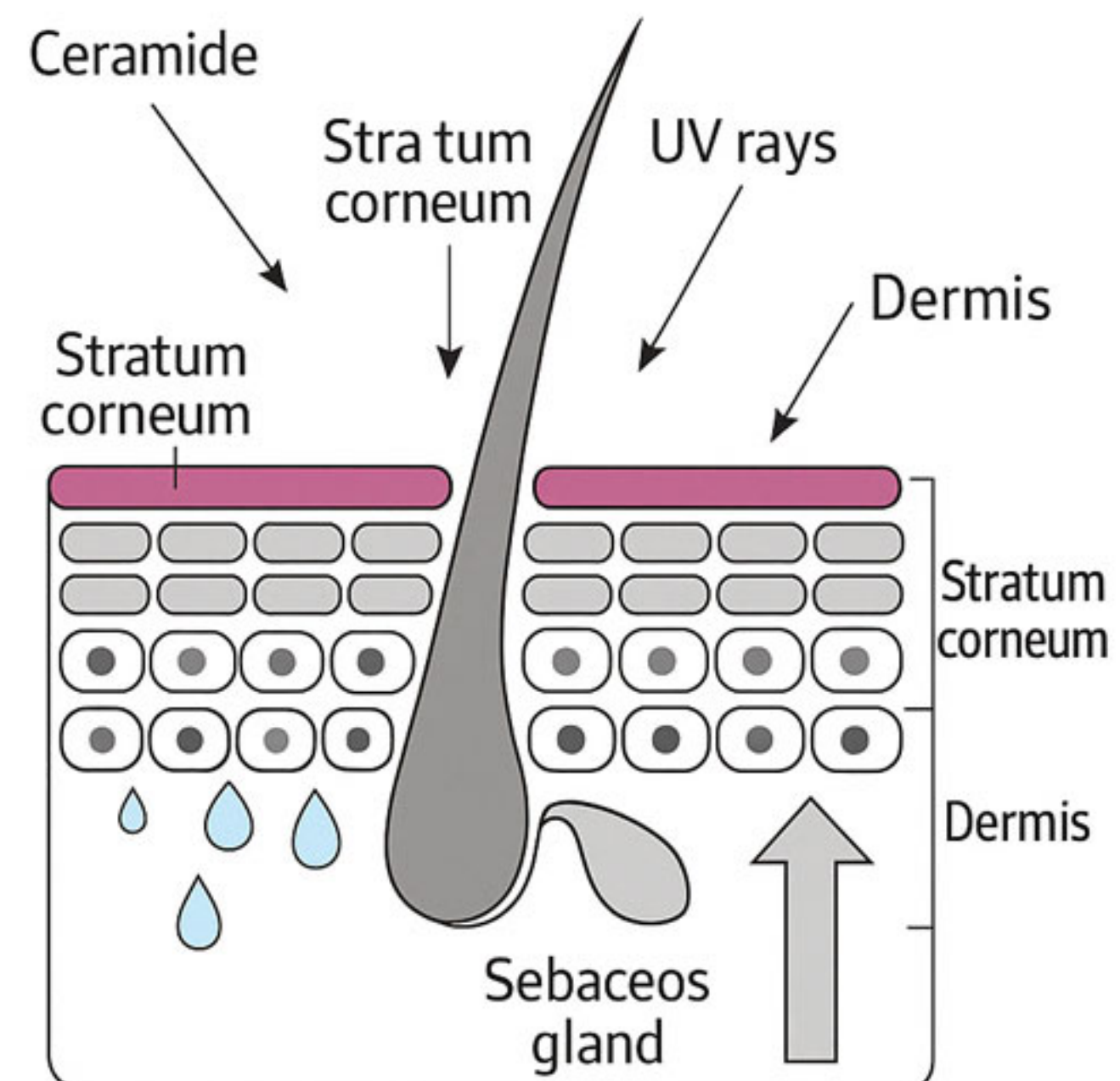



Healthy skin structure and dermis layer

What is Polynucleotide **PN**?

Similarity between Human DNA bases and Salmon DNA base fragments

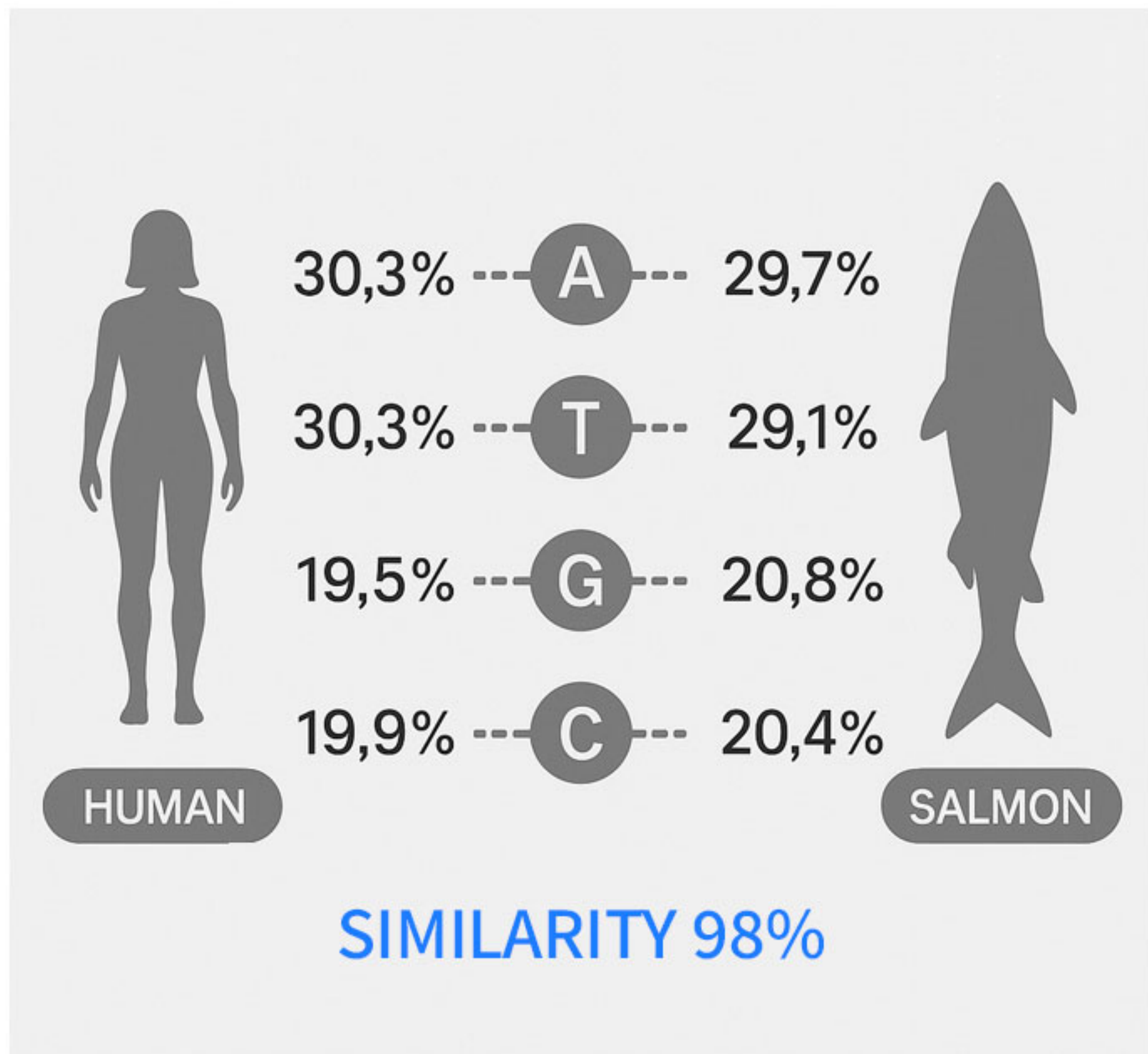


Polynucleotide (PN) injections deliver ultra-refined DNA fragments sourced from salmon, scientifically validated to activate the skin’s intrinsic regenerative mechanisms— particularly effective for delicate and sensitive skin. By stimulating collagen synthesis and restoring dermal integrity, PN therapy enhances skin elasticity, firmness, and overall texture with remarkable precision.

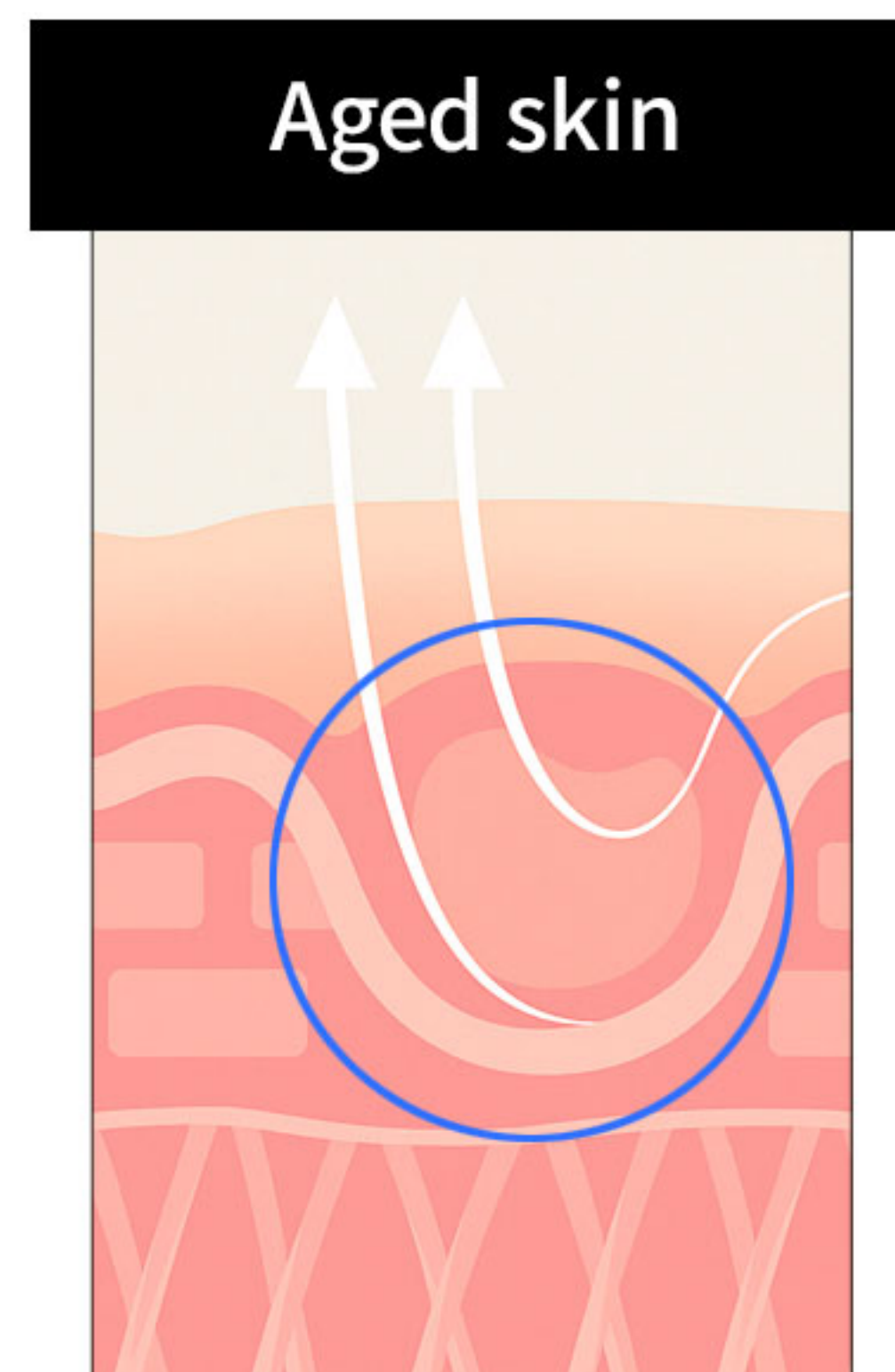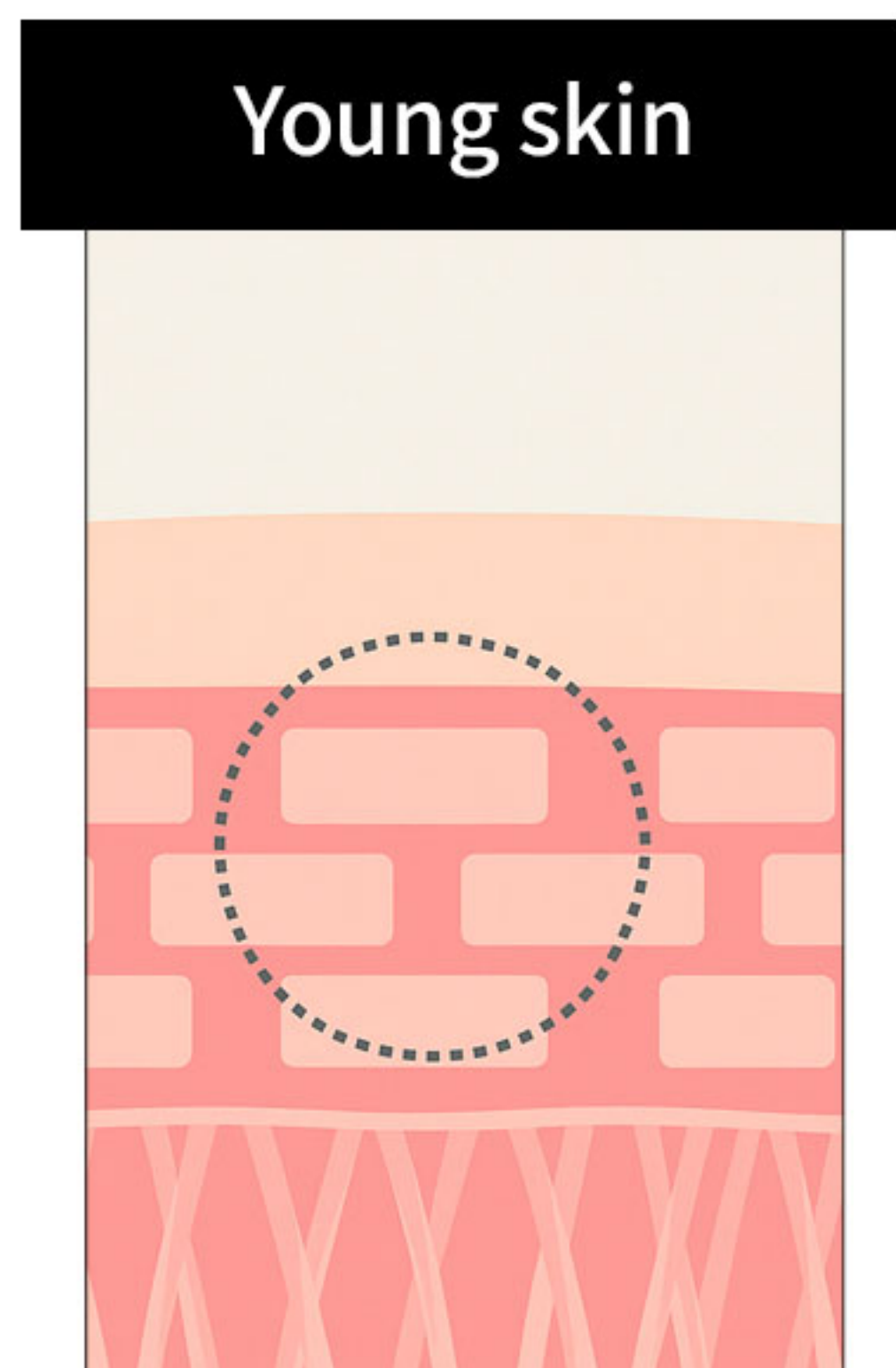

Advantages of Polynucleotide

- ▶ Increases VEGF: Boosts circulation to restore damaged skin
- ▶ Activates Fibroblasts: Promotes collagen and protein production
- ▶ Regenerates ECM: Restores firm, healthy skin structure
- ▶ High-Purity Formula: Patented refinement, zero immune reaction

REDJUR eyes treatment area

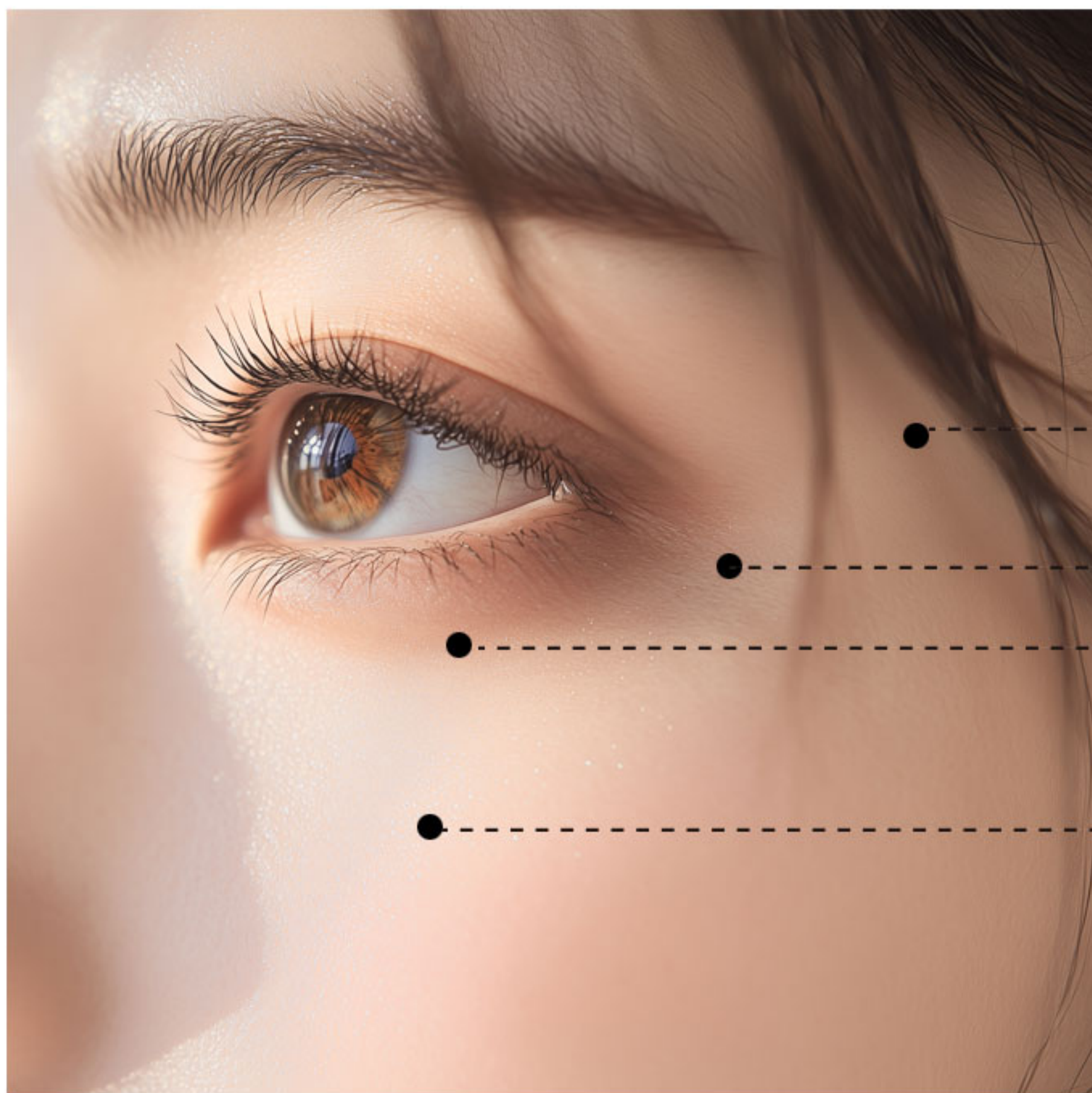

Temple

Crow's Feet

Tear trough

Midface




INJECTION SPACING

- 0.5-2 cm between each injection site
- Inject very small amounts (0.01-0.02 ml per spot)
- Typically done in a series of 3-4 sessions, spaced 3-4 weeks apart.

Application Method

Can be used via mesotherapy, micro-needling, or manual techniques in clinical or professional skincare settings.

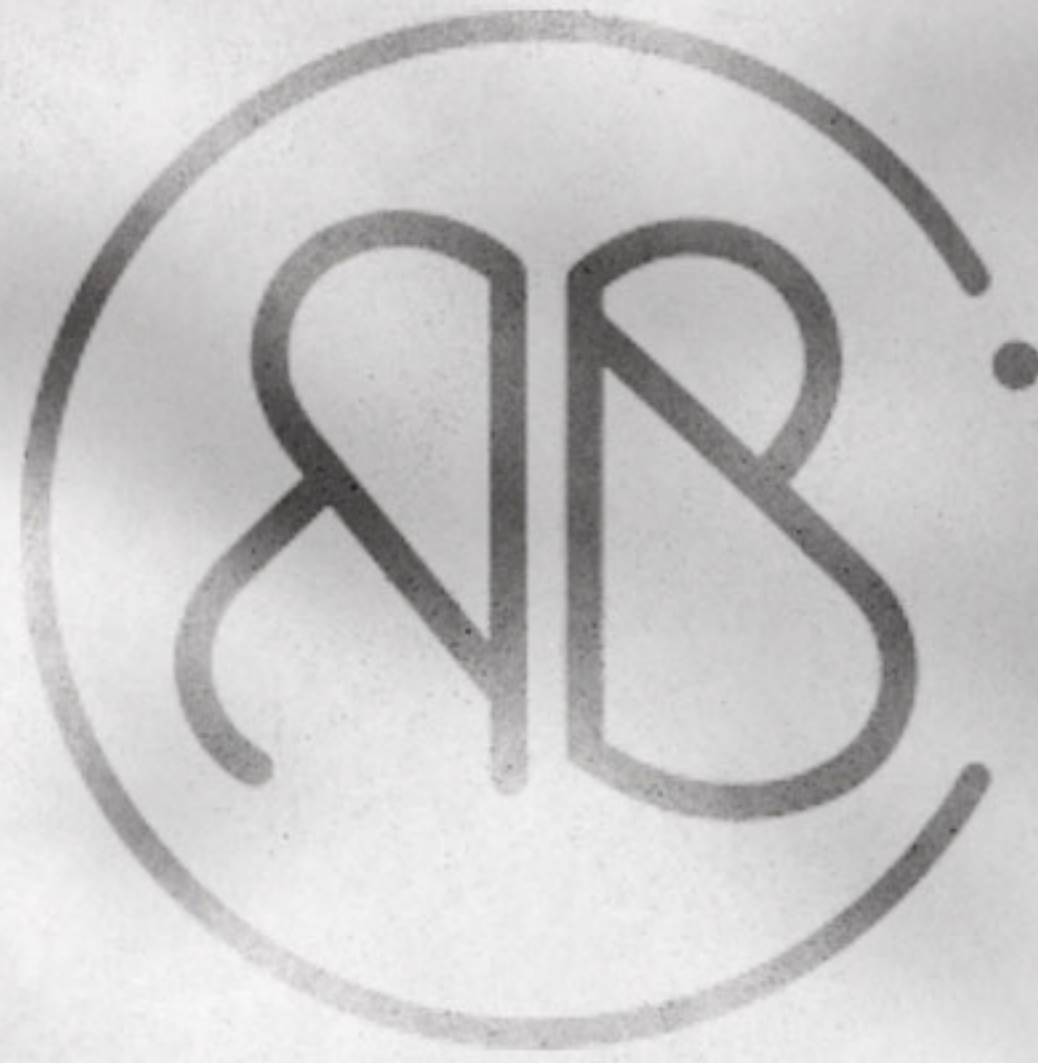

RED BIO COMPANY

RED BIO COMPANY Co., Ltd.

Address B dong 1002, 1003, 1004 Iljik-ro 43, Gwangmyeong-si, Gyeonggi-do, Republic of Korea

Factory 514, 27 Dunchon-daero 457beon-gil, Jungwon-gu, Seongnam-si, Gyeonggi-do, Republic of Korea

Tel +82-2-6914-6680 **Fax** +82-2-6914-6681

E-mail info@redbiocp.com www.redbiocp.com