



Rejuva
Healer S

HA + PDRN + NAD

2.5 mL x 3 syringes



HA + NAD + PDRN

TRIPLE SCIENCE FOR SKIN REGENERATION



Phace Global Co., Ltd

You Wanted
a Booster for All..?

NAD+



We Made It.

Triple-Action Formula for Complete Skin Renewal

PDRN

REGENERATE
FROM WITHIN

NAD+

RECHARGE
SKIN CELLS

HA

DEEP
HYDRATION

Hydration

2000-2010

Hydration & Texture

The Birth of first
HA Skinboosters

Healing

2011-2015

Repair & Healing

The Rise of
PDRN/PN Boosters

2016-2020

Multi-Functional Formulas

Growth factors,
Peptides, Amino acids

2021-2023

Regeneration & Soothing

Exosome-based
Boosters hit the market

Cellular Energy

2024-2025

Cell power & Collagen boost

New Generation
NAD+ and NMN boosters



The New Generation Booster

Unites the Best of All Skin Booster Eras
for Results Beyond Time

Rejuva Healer S

What Results Will You See?



Hydration & Pore-Tightening



Reduction in Fine Lines & Dullness



Overall Revitalized, Radiant skin



Improved skin Texture & Elasticity

HA + NAD + PDRN

2.5 mL X 3 Syringes

What's Inside Rejuva Healer S?

Uses only **High-Purity, Non-Animal** origin HA

Balanced Strength
17mg/mL

Ultra-Pure
99.8%

PH Balanced
6.8 – 7.5



Deep Hydration, Real Results

Non-Crosslinked HA 17mg/mL x 1 Syringe

*Internal Clinical Study Report, Rejuva Healer S Performance and Safety Results, 2024.

What's Inside Rejuva Healer S?

Formulated with 2% of the **Purest, High-Potency PDRN**

High-Purity DNA Extract

(>95%)

Non-Immunogenic
Biocompatible & Gentle

Laboratory-Tested
100% safe



*Internal Clinical Study Report, Rejuva Healer S Performance and Safety Results, 2024.

What's Inside Rejuva Healer S?

High-Potency NAD+: The Anti-Aging Supercharger

Texture & Radiance

Smoother and More Even Skin

Boosts Cellular Energy

Stimulates ATP Production

Slow Aging

Restoring Youthful Cells

Long-Lasting

Cell Rejuvenation

How NAD+ Works?

NAD+ (Nicotinamide Adenine Dinucleotide) is a **Vital Coenzyme** found in every cell that powers energy production and supports cellular repair. NAD+ fuels cellular energy by supporting mitochondrial function. It activates DNA repair, enhances cell regeneration, and boosts metabolism.



What is NAD+ derived from?

NAD+ is a Biologically active and Lab-grade material, typically derived from Nicotinamide (Vitamin B3), ensuring purity, safety, and optimal cellular effectiveness.

Is NAD+ safe for all skin types?

Yes, NAD+ is non-immunogenic, biocompatible, and well-tolerated, making it suitable for all skin types and professional skin booster treatments.

How does NAD+ work with PDRN and HA in Rejuva Healer S?

NAD+ energizes cells, PDRN stimulates regeneration, and HA hydrates deeply, working together to deliver comprehensive skin rejuvenation at the cellular level.

What Makes Rejuva Healer S the Real Game changer?


One Booster. Total Care.

Forget layering multiple products — one formula to hydrate, repair and recharge your skin.



Powered by What Works

HA for deep hydration. PDRN for regeneration. NAD+ for energy. All clinically proven. All in 1 syringe.

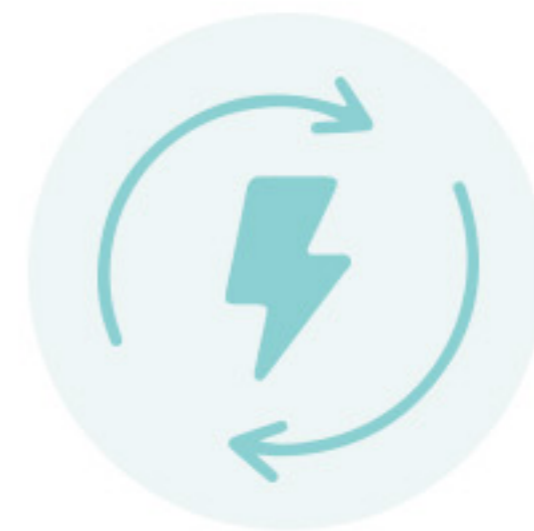

Zero Downtime, All Results

Walk out with radiant, refreshed skin — no redness, no irritation, no recovery time.



Trusted by Professionals

Used in clinics worldwide for its safety, purity, and real results. New, but already trusted.




Rejuva
Healer S


New-Era Skin Booster

What Makes Rejuva Healer S Unique:

Rejuva Healer S	VS	Kia***	Reju****
17 mg/mL	HA	20 mg/mL	20 mg/mL
2% High-Potency	PDRN	5 mg/mL	Not included
✓ Maximum Potency	NAD+	✗ Not included	✗ Not included
2.5 mL x 3 Syringes	Volume	2.2 mL x 3 Syringes	2.5 mL x 3 Syringes
CE-Grade, GMP, Sterile	Manufacturing Standards	Not specified	Not specified
Hydration & Radiance Firmness Pore Tightening Cellular Regeneration	Expected Benefits	Hydration Regeneration	Hydration Skin Smoothness

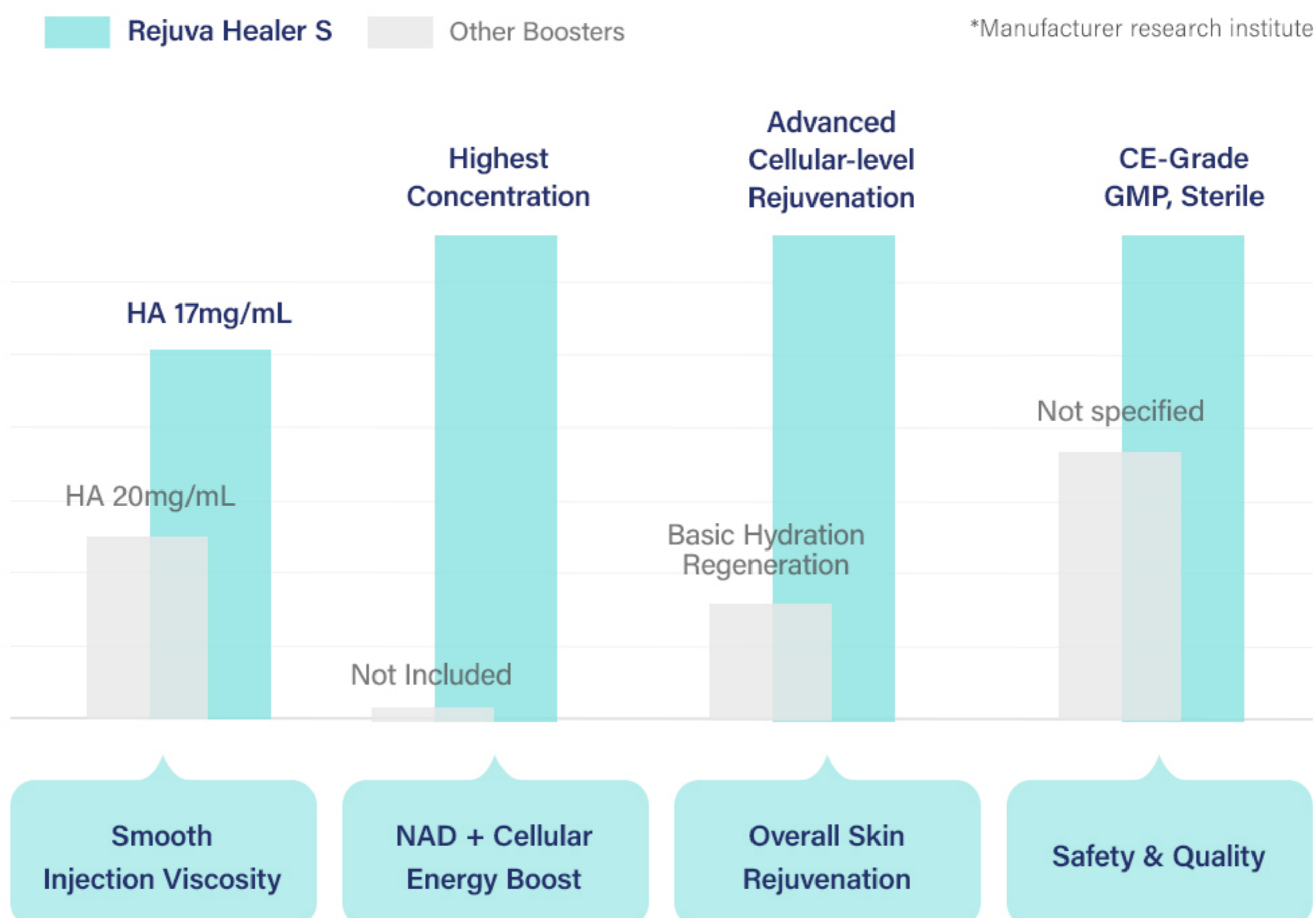

Injection Areas & Technique:

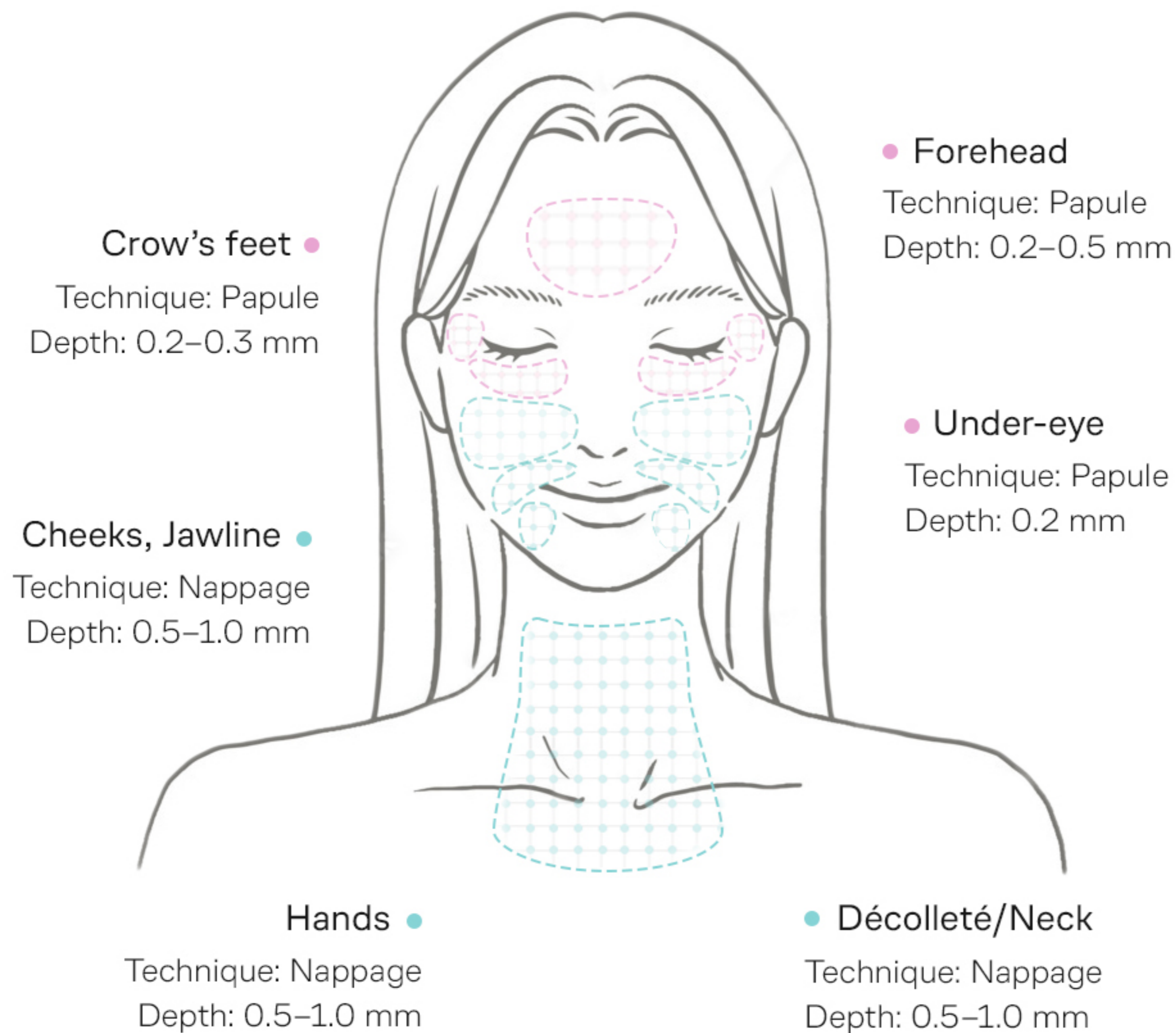

A. Papule Technique

- Depth: 0.2–0.5 mm (SP)
- Angle: 10–15°
- Volume per point: 0.01–0.02 mL
- Creates small, visible bumps that disappear within 24–48 h
- Best for: under-eye, forehead, crow's feet, fine lines


B. Nappage Technique

- Depth: 0.5–1.0 mm (SP–MD)
- Angle: 30° with rapid, light injections
- Volume per point: 0.01–0.02 mL
- Spacing: 0.5–1 cm apart
- Best for: cheeks, chin, jawline, neck, décolleté, hands

NEEDLE SIZES

30G × 4mm needle for precise, comfortable mesotherapy injections
32G needles are recommended for delicate areas like under the eyes

Recommended Treatment Schedule:

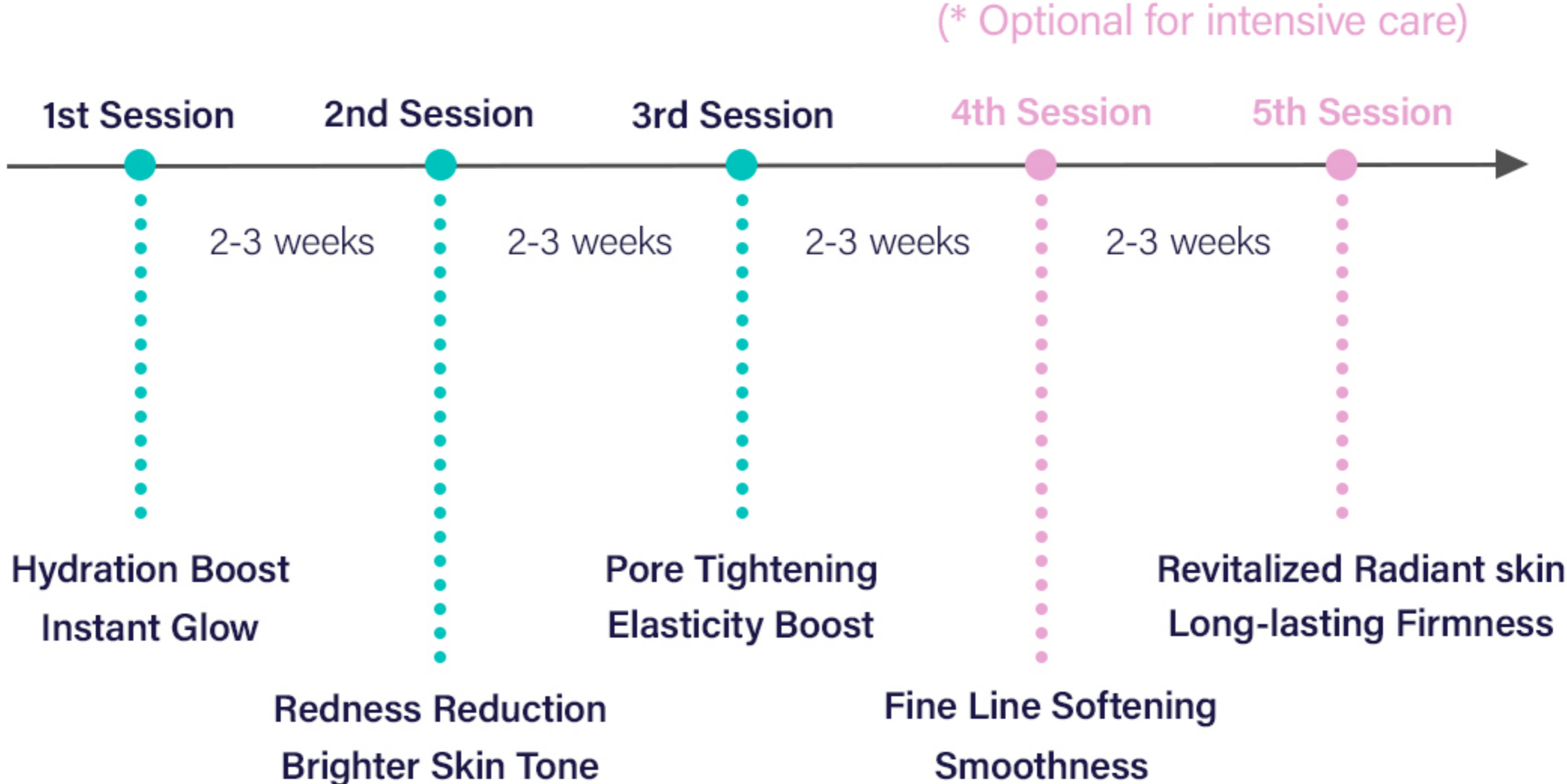

Composition:

- HA 17 mg/mL + PDRN 2% + NAD+

Storage & Precautions:

- Store in a cool, dry place (1-30°C), away from direct sunlight
- Avoid contact with eyes or mucous membranes
- For external/professional use only
- Exp.Date: 36 months from the date of manufacture

Packaging: 2.5mL x 3 Syringes

*Clinical Insights from Manufacturer-Conducted Research





Before



After



Rejuva Healer S

After 1 session:


- Hydration Boost
- Instant Glow
- Brighter Skin Tone



Before



After

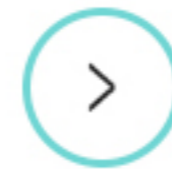

After 3 sessions:


- Hydration Boost
- Brighter Skin Tone
- Pore Tightening
- Elasticity Boost
- Fine Line Softening



Before



After

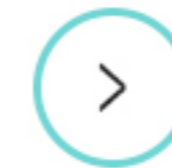

After 5 sessions:

- Hydration Boost
- Brighter Skin Tone
- Pore Tightening
- Elasticity Boost
- Fine Line Softening
- Skin Firming
- Overall Revitalized Radiant skin

New-Era Skin Booster



Distributor: Esta Global Co., Ltd
Manufacturer: Phace Global Co., Ltd